



For complete user manual, visit [air2zed.com](http://air2zed.com)


## PACKAGE CONTENTS:

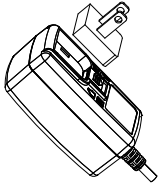


Power Supply

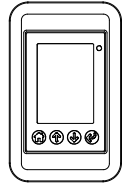


1 Transmitter

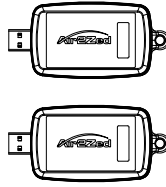

2 Receivers

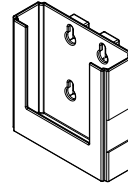

Mounting Cradle

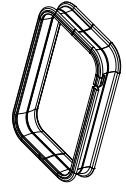


Cable-Retainer


## 3 SIMPLE STEPS:

① Connect Receiver to Computer


② Plug Sensor into Transmitter

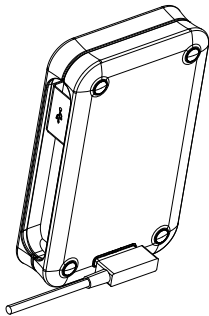

Charge transmitter for 2 hours before first use

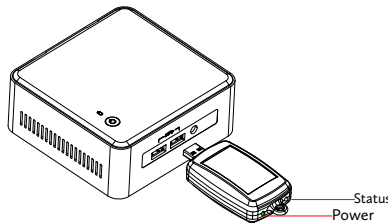

Wait for POWER Led to light up (may take up-to 1 minute)

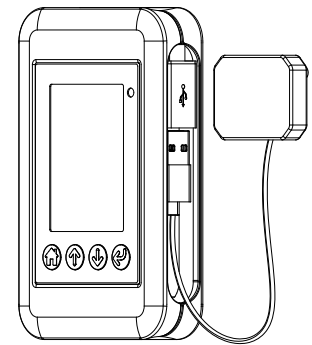

Press any key wake up the transmitter

③ Pair transmitter with receiver



Use UP/DOWN buttons to select from available receivers



Press ENTER to initiate pairing with receiver



Paired receiver is displayed  
Sensor is now ready for use

(STATUS led on receiver lights up)

- No additional software required
- Optionally wrap excess sensor wire around supplied cable-retainer for compact set-up
- Mounting cradle can be fixed to wall or around x-ray pole, providing convenient storage for transmitter
- Scan QR code below or visit our website for complete user-manual and additional resources



# ZINNOVI

for support: contact 847-430-4999 or e-mail [support@air2zed.com](mailto:support@air2zed.com)

135 W. Central Road  
Suite 204  
Schaumburg, IL

